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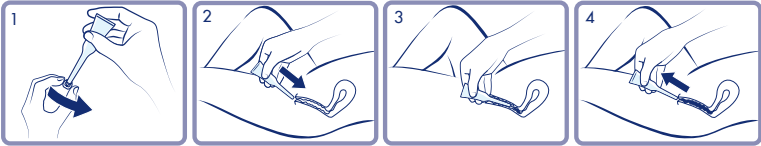
🇮🇹

### EN Instruction for use of Ritex KINDERWUNSCH

Ritex KINDERWUNSCH Conception Lubricant is a medical device for application in the vaginal area to support couples in the desire to have children through natural conception. With its excellent glide properties, it effectively compensates for a lack of vaginal moisture (e.g. as caused by stress, medication, etc.). Clinical in-vitro studies have shown that the Ritex KINDERWUNSCH Conception Lubricant preserves characteristics of sperm which are important for successful conception, such as motility, vitality, morphology and integrity of DNA, very well.

#### But what's so great about the Ritex KINDERWUNSCH Conception Lubricant?

Most conventional lubricants are adapted to the vaginal pH value during non-fertile days and therefore are much too acidic for sperm. Furthermore, the osmolality of many lubricants is very high, so that the activity of sperm is significantly reduced or even completely removed. Not forgetting the fact that there are also a large number of chemical substances in many other lubricants that act in a similar way to hormones, which can mislead the sperm in their search for the egg. The Ritex KINDERWUNSCH Conception Lubricant has been developed so that the crucial parameters of pH value and osmolality are preserved in a naturally given measure. The viscosity is similar to that of the cervical mucus on fertile days so that the sperm are supported in their forward motion. We examined sperm irritation caused by an effect similar to hormones and eliminated this. In addition, it's free from preservatives and thus particularly gentle on sperm.

#### When should you use the Ritex KINDERWUNSCH Conception Lubricant?

The lubricant should be used during a woman's fertile days (approx. 6 days per cycle: 4 days before ovulation, the day of ovulation itself and one day after ovulation). The highest probability for fertilisation is on the day of ovulation.

#### Instructions for use

The lubricant is effective immediately and can be applied immediately before intercourse. Before first use, tear open the perforation of the protective pouch. Remove an applicator and reseal the protective pouch with the zipper carefully. Remove the protective cap from the applicator (1) and discard. Please do not use the applicator if the protective cap is damaged or missing. The faulty applicator must be disposed of and a new one must be used instead. Please use the applicator in a lying position so that the lubricant can be optimally introduced. Hold the applicator in the middle area of the tube with your thumb and fingers and then introduce the tip and carefully into the vagina (2). Squeeze the middle of the tube several times in order to remove the lubricant (3). Then pull the applicator out of the vagina and keep squeezing the tube so that no vaginal secretion is sucked in (4). Please dispose of the used applicator with domestic waste. Ritex KINDERWUNSCH Conception Lubricant can leave stains on textiles which may prove difficult to wash out. Lubricant must not be allowed to come into contact with the eyes, as the lubricating film can cause temporarily cloudy vision. Keep out of the reach of children.

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