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**KINDERWUNSCH**  
**Conception Lubricant**  
**Supports natural conception**

Sperm friendly  
Clinically tested (in vitro)  
Dermatologically tested  
Free from preservatives

<b><span>EN</span></b> Explanation of symbols	<b><span>DE</span></b> Symbolerklärung	<b><span>FR</span></b> Signification des symboles	<b><span>IT</span></b> Legenda dei simboli
<b><span>ES</span></b> Explicación de los símbolos	<b><span>SV</span></b> Förklaring av symboler	<b><span>NL</span></b> Verklaring van symbolen	<b><span>PL</span></b> Objasnienie symboli
<b><span>CZ</span></b> Vysvětlivky symbolů	<b><span>SK</span></b> Vysvetlenie symbolov	<b><span>RO</span></b> Simboluri	<b><span>RU</span></b> Пояснения к символам
<b><span>VN</span></b> Giải thích các biểu tượng	<b><span>TR</span></b> 符号说明		

<b><span>IT</span></b> <b><span>EN</span></b> Please note the detailed instructions in this package leaflet. <b><span>DE</span></b> Bitte beachten Sie die ausführlichen Hinweise auf diesem Beipackzettel. <b><span>FR</span></b> Prière de se conformer aux instructions détaillées fournies dans le présent mode d'emploi. <b><span>IT</span></b> Si prega di rispettare le indicazioni dettagliate fornite nel foglietto illustrativo. <b><span>ES</span></b> Observe las indicaciones detalladas de este folleto explicativo. <b><span>SE</span></b> Se de utföriga anvisningarna på denna bipacksedel. <b><span>NL</span></b> Let op de gedetailleerde informatie op deze bijsluiter. <b><span>PL</span></b> Należy zapoznać się ze szczegółowymi instrukcjami na ulocie dołączonej do opakowania. <b><span>CZ</span></b> Rďte se podrobnými pokyny uvedenými v tomto příbalovém letáku. <b><span>SK</span></b> Prečítajte si detailné pokyny na prílohu príbalového letáku. <b><span>RO</span></b> Vă rugăm să aveți în vedere instrucțiunile detaliate din acest prospect. <b><span>RU</span></b> Пожалуйста, придерживайтесь подробных указаний, приведенных в данном листке-вкладыше. <b><span>VN</span></b> Vui lòng lưu ý vào sách thông tin chi tiết trong tờ thông tin kèm theo này. <b><span>TR</span></b> Lütfen dikkatlembelge sayfadındaki ayrıntıları okuyunuz. <b><span>AR</span></b> يرجى احترام التعليمات المفصلة الواردة في هذه النشرة المرفقة بالمعوية.	<b><span>EN</span></b> Expiry date. Do not use applicators past their expiry date. <b><span>DE</span></b> Verfalldatum. Applikatoren nicht benutzen, wenn Verfalldatum überschritten ist. <b><span>FR</span></b> Date de péremption. Ne pas utiliser les applicateurs si la date de péremption est dépassée. <b><span>IT</span></b> Data di scadenza. Non utilizzare gli applicatori oltre la data di scadenza. <b><span>ES</span></b> Fecha de caducidad. No utilice aplicadores que hayan excedido la fecha de caducidad. <b><span>SE</span></b> Utgångsdatum. Använd inte applikatorerna om utgångsdatumet har passerats. <b><span>NL</span></b> Vervaldatum. Gebruik applicatoren niet wanneer de vervaldatum verstreken is. <b><span>PL</span></b> Data ważności. Nie używaj aplikatorów po upływie daty ważności. <b><span>CZ</span></b> Datum expirace. Nepoužívejte aplikatory, pokud výpřelo datum expirace. <b><span>SK</span></b> Dátum expirácie. Aplikatory nepoužívaťe, ak už uplynú dátum expirácie. <b><span>RO</span></b> Data expirării. Nu folosiți aplicatoarele dacă data expirării este depășită. <b><span>RU</span></b> Срок годности. Не использовать аппликаторы по истечению указанного срока годности. <b><span>VN</span></b> Hạn sử dụng. Không sử dụng các ống đã hết hạn sử dụng. <b><span>TR</span></b> 到期日期。不得使用已过期的涂沫器。
---	--

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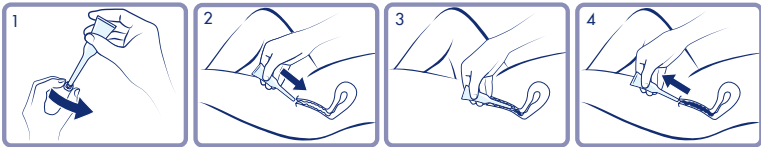
## (EN) Instruction for use of Ritex KINDERWUNSCH

Ritex KINDERWUNSCH Conception Lubricant is a medical device for application in the vaginal area to support couples in the desire to have children through natural conception. With its excellent glide properties, it effectively compensates for a lack of vaginal moisture (e.g. as caused by stress, medication, etc.). Clinical in-vitro studies have shown that the Ritex KINDERWUNSCH Conception Lubricant preserves characteristics of sperm which are important for successful conception, such as motility, vitality, morphology and integrity of DNA, very well.

### But what's so great about the Ritex KINDERWUNSCH Conception Lubricant?

Most conventional lubricants are adapted to the vaginal pH value during non-fertile days and therefore are much too acidic for sperm. Furthermore, the osmolality of many lubricants is very high, so that the activity of sperm is significantly reduced or even completely removed. Not forgetting the fact that there are also a large number of chemical substances in many other lubricants that act in a similar way to hormones, which can mislead the sperm in their search for the egg. The Ritex KINDERWUNSCH Conception Lubricant has been developed so that the crucial parameters of pH value and osmolality are preserved in naturally given measures. The viscosity is similar to that of the cervical mucus on fertile days so that the sperm are supported in their forward motion. We examined sperm irritation caused by an effect similar to hormones and eliminated this. In addition, it's free from preservatives and thus particularly gentle on sperm.

### When should you use the Ritex KINDERWUNSCH Conception Lubricant?

The lubricant should be used during a woman's fertile days (approx. 6 days per cycle: 4 days before ovulation, the day of ovulation itself and one day after ovulation). The highest probability for fertilisation is on the day of ovulation.

### Instructions for use

The lubricant is effective immediately and can be applied immediately before intercourse. Before first use, tear open the perforation of the protective pouch. Remove an applicator and reseal the protective pouch with the zipper carefully. Remove the protective cap from the applicator (1) and discard. Please do not use the applicator if the protective cap is damaged or missing. The faulty applicator must be disposed of and a new one must be used instead. Please use the applicator in a lying position so that the applicator can be optimally introduced. Hold the applicator in the middle area of the tube with your thumb and fingers and then introduce the thin end carefully into the vagina (2). Squeeze the middle of the tube several times in order to remove the lubricant (3). Then pull the applicator out of the vagina and keep squeezing the tube so that no vaginal secretion is sucked in (4). Please dispose of the used applicator with domestic waste. Ritex KINDERWUNSCH Conception Lubricant can leave stains on textiles which may prove difficult to wash out. Lubricant must not be allowed to come into contact with the eyes, as the lubricating film can cause temporarily cloudy vision. Keep out of the reach of children.

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